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ⓐ Important Instructions – please read carefully before use

The Special Teat Range from NUK – For children with a cleft lip or cleft palate. NUK has developed a special teat range, in the original NUK orthodontic shape, for children with a cleft lip or cleft palate. These teats are designed as a feeding aid. It is precisely when feeding and drinking aids are required that physiological and psychological needs are often not met. For newborn babies affected by these conditions ingestion can be associated with considerable problems. The reason for this is the inability of these infants to generate an intraoral vacuum, thus making sucking impossible. This is not only the case for breastfeeding, but also for bottle feeding using commercially available teats. It is extremely difficult for these infants to suck, and in many cases it can be impossible. To help these infants learn to suck with some degree of success, NUK has developed a

range of special teats based on scientific knowledge.* These teats are based on the specific, anatomical-morphological conditions of the alveolar region of children born with clefts - both in form and function.

The NUK Cleft Palate Teat – without feed hole

This teat has been designed so that the large mouthpiece of the teat seals the cleft in the palate. **Important Warning: The feed hole in the teat must be manually made for each individual infant. This should be done, under medical supervision, according to the position of the cleft in the infant's palate (see fig. 1-4).** A feeding hole can be made in the teat using a hot needle. However, it is more hygienic to push a toothpick from the inside against the teat and then to cut off the tip of both toothpick and teat. The feeding hole should be made on the outer edge of the mouthpiece on the side furthest away from the cleft. It should not be made on the under side, as otherwise the hole will be covered by the tongue. One of the most important characteristics of the NUK Cleft Palate Teat is that it promotes the natural sucking reflex and milking action while feeding. With every mouthful that the infant takes the large arched roof of the teat presses against the palate, which seals the cleft allowing the milk to flow into the fold of the cheek. This stops the milk from entering the respiratory tracts, preventing the infant from choking. Milk is therefore mixed with saliva, which is important for the subsequent digestion process.

The NUK Cleft Palate-teat Art.-No. 10.107.047 easily fits on NUK First Choice and NUK New Classic Bottles. The NUK Cleft Palate teat Art.-No. 10.107.001 easily fits on any Standard Bottles with screw thread.

The NUK Cleft Lip Teat with a Medium Feed Hole (for milk)

For children with a cleft lip and those with a cleft palate where a plate has been fitted to the roof of the mouth. This teat was developed as a sucking aid for children with unilateral (one-sided) clefts and dental plates for feeding, and in exceptional cases can be used for children with bilateral clefts and palates. These teats close the oral cavity to the atmosphere by covering the outer cleft area. This enables the infant to generate an intraoral vacuum.

The part of the teat which covers the upper lip is particularly soft. When the infant begins to suck this part seals against the upper lip preventing further air flowing through the cleft. This teat is used in cases where the cleft palate has already been sealed by a dental plate. For children born with cleft lip and cleft palate, this teat permits the correct sucking action while feeding from the bottle, and at the same time fulfils the baby's inherent need to suck.

In contrast to other feeding methods, when using the NUK Cleft Lip Teat the otherwise frequently observed symptoms, such as choking, spitting out and even vomiting, are considerably reduced. There are also no detrimental preoperative, orthopaedic corrective measures required.* The NUK Cleft Lip Teat was developed in cooperation with scientists.* Form and function have been successfully tested in many hospitals worldwide. This teat is comparable with the original NUK teat and does not have the appearance or properties of a prosthesis. It has properties that are particularly important for this type of special teat, allowing it to form a seal over the cleft lip.

The NUK Cleft Lip teat Art.-No. 10.107.046 easily fits on NUK First Choice and NUK New Classic Bottles. The NUK Cleft Lip teat Art.-No. 10.107.003 easily fits on any Standard Bottles with screw thread. **Important:** Breastfeeding is best for babies. **Preparation:** When preparing feed or teas, always follow the temperature specifications of the feed manufacturer. To retain vitamins and nutrients, the temperatures should not exceed 60 °C. Do not microwave bottles filled with feed or water (risk of explosion/material damage). Uneven heating may result in scalding! Never heat sealed bottles as they may explode! Please softly squeeze drinking hole before each use to ensure that it is open. Only administer food and drink in the quantity and consistency recommended by the manufacturer. **Cleaning bottles, teats and accessories:** Before first use place in boiling water for 5 minutes, use sufficient water to avoid damage. For hygiene reasons, clean thoroughly using mild washing up liquid and sterilise before every further use. We recommend using the NUK Steriliser for sterilisation. Certain foodstuffs (e.g. carrot juice) may discolour the

material or affect its odour (e.g. fennel tea). This has no effect on the fitness for use. Teats and bottles should not be cleaned in a dishwasher – dishwasher detergents can cause material damage! For sterilisation in a microwave oven we recommend using the NUK Microwave Steriliser (please follow the instructions of the steriliser/the device!).

For your child's safety and health: WARNING!

Always use this product with adult supervision. Never use feeding teats as a soother. Continuous and prolonged sucking of fluids will cause tooth decay. This applies in particular to sweetened drinks and fruit/vegetable juices. **Always check food temperature before feeding. Always check the teat for signs of excessive wear by pulling the teat in all directions before each use especially when the child has teeth. To avoid the danger of small parts being swallowed by the child, replace the teat when sticky from use, visibly damaged or showing signs of tears or bites.** For safety and hygiene purposes, replace the teat after 1-2 months use. Do not expose the teat to direct sunlight/heat or leave in disinfectants (sterilising solution) for longer than the stated time as this can damage the teat. Store hygienically in a dark and dry place. Do not dip in medicine! Produced from natural rubber latex which may cause allergic reactions. NUK Product complies with EN 14350. Please retain distributor's address, article number and LOT number for possible future reference. Distributed in the UK by: MAPA Spontex UK Ltd., Berkeley Business Park, Wainwright Road, Worcester WR4 9ZS, Great Britain; www.nuk.co.uk Distributed in Ireland by: Intrapharma Consumer Health, Magna Business Park, Citywest Road, Dublin 24, Ireland, +353 1 463 2424 * Source: Dipl.-Bio. Susanne Hummel, Institut für Anthropologie der Universität Goettingen. Published: Eine Saughilfe für Sauglinge mit Lippen-Kiefer-Gaumen-Spalte/Fortschritte der Kieferorthopädie, (1987), 26-33 (Nr.1)

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MAPA GmbH Industriest. 21-25 27404 Zeven/Germany www.nuk.com